

Vantive

NEW

WE'RE MAKING YOUR LIFE ON PD A LITTLE EASIER*

Key enhancements to Vantive's
Extraneal and Dianeal range

vantive.asia

WHAT'S

NEW

Our Extraneal and Dianeal peritoneal dialysis fluid packaging has been redesigned, to help make PD a little easier for you.

The composition of our fluids remains the same, and no significant changes are required by you.

All enhancements are compatible with how you use our products.

Your PD clinic may contact you to discuss these changes further.

MACHINE BAGS


MACHINE BAG (APD) ENHANCEMENTS

Key enhancements to Vantive's Machine (APD) range of products.



Easy-open overpouch

Our updated bag overpouch is now easier* to open. The lighter-weight pouch features dual, deeper, pre-cut tear-slits, making it easier than ever to open.

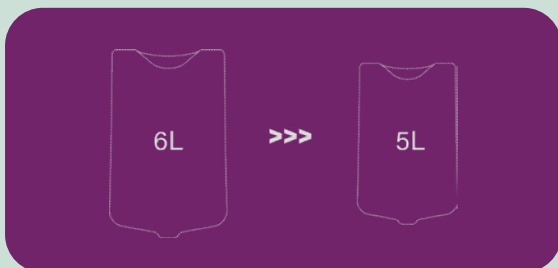

Lighter, more compact 5 litre APD bag

A new lighter and more compact 5L APD bag replaces the 6L bag, reducing physical burden and waste for many patients



Easier product identification

New coloured labels replace coloured tape to help identify products and concentrations. The colour scheme remains unchanged. We've also added a solid star to identify APD bags.



Strong, reinforced box construction

Our new boxes are sturdy, with reinforced construction. They're slightly taller and slimmer, optimised for efficient stacking and storage, and designed to better protect the fluid bags.



Easy pull tabs

The new boxes feature handy green pull tabs on both ends, making it quick and easy to open the boxes

IMPORTANT APD CHANGE

Reversed port folding memory

The port on our new bags now folds downwards instead of upwards. Before you connect your bag, to maintain aseptic technique, place the bag label facing down to prevent the port from folding under.



LABEL UP



LABEL DOWN



Always place the bag label down to prevent the port from folding under.

MANUAL BAGS

MANUAL BAG (CAPD) ENHANCEMENTS

Key enhancements to Vantive's Manual (CAPD) range of products.



Simple CAPD bag setup

Hanging the bag is now smoother, thanks to an improved* punch hole design that's easier to place on a hook



Easy pull tabs

The new boxes feature handy green pull tabs on both ends, making it quick and easy to open the boxes




Easy-open overpouch

Our updated bag overpouch is now easier* to open. The lighter-weight pouch features dual, deeper, pre-cut tear-slits, making it easier than ever to open.



Strong, reinforced box construction

Our new boxes are sturdy, with reinforced construction. They're slightly taller and slimmer, optimised for efficient stacking and storage, and designed to better protect the fluid bags.

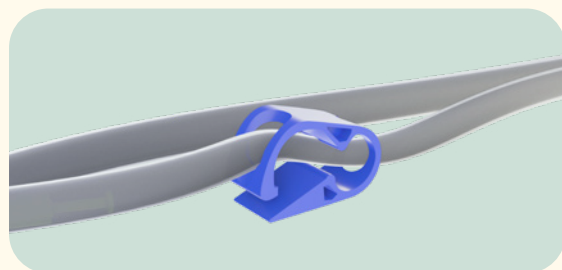

Improved* clamping

The improved clamp is now easier to open and close allowing you to manage the process confidently



Easier product identification

New coloured labels replace coloured tape to help identify products and concentrations. The colour scheme remains unchanged.

Vantive

All enhancements are compatible with how you currently use our products.

For more help with these changes, please contact your PD clinic.

Australia
Vantive Pty Ltd

1 Baxter Drive
Old Toongabbie
NSW 2146

☎ 1800 920 133

🌐 vantive.asia

New Zealand
Vantive Limited

33 Vestey Drive
Mt Wellington
Auckland 1060

☎ 0800 436 770

🌐 vantive.asia

*Improved from existing Vantive PD solutions

ANZ-RC00-250054